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ABPM 6100 Software Installation Guide

Minimum PC System Requirements:

100% IBM Compatible, Windows 95, 33 MHz 486DX processor, 16 MB of RAM, 30 MB of hard drive space

Serial port for communication with unit.

Welch Allyn ABPM Monitor Requirements:

All ABHPM 6100 units, All White ABPM/QuietTrak/Tenso24 Units
Black ABPM/QuietTrak & Gray Tenso24 units with internal monitor software, version 4.2 or higher

Installing ABPM 6100 Software:

1. Insert the ABPM 6100 CD into your CD-ROM drive.
2. If the CD does not automatically Start, then Click Start, Run, and type d:\autorun.exe (where d: is the CD-ROM drive)
3. Follow the on screen instructions.

Installation Problems:

Please refer to the PC based Operator's Manual, provided in Adobe Acrobat format on the Installation CD. Adobe Acrobat Reader can be installed directly from the CD, if required. A list of common installation problems and technical service centers can be found there

If unable to access the Operator's Manual or for additional Technical Service, contact your authorized Welch Allyn dealer or access www.welchallyn.com on the Internet and click on "Contact Info". A detailed list of Worldwide Service Centers can be found there.

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