

User's Manual of Face Shield

Face Shield is non-sterile disposable anti-fog Face Shield.

- Brand: Refine
- Product name: Face Shield
- Material: PET 80% PU sponge 20%.
- Model: B
- Composition: composed of protective cover and fixing device made of polymer material. Non-sterile, single use.
- Intended use : Face Shield is common eye protection equipment, which are used for protecting against droplets and splashes of liquids. Recommended industry: Facility Sanitation, Food Processing, Food Safety.
- Product instruction: The product and the protective packaging should be checked before using. Stop using it if there is any damage.
- Precautions:
 - One-time use only.
 - Use in accordance with the instructions.
 - Never use it when damages are found.
 - Stay away from chemicals.
 - The eye protector is not intended to protect against high speed particles.
- Storage: Please store in clean, dry and ventilated indoor place, with relative humidity (10%-93%), Atmospheric pressure for storage (70kPa-106kPa), temperature (-20°C- +40°C), avoid corrosive gas.

Warning: a. Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals; b. Scratched or damaged oculars should be replaced; c. Eye-protectors against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer. d. These protectors are intended for indoor use where no optical radiation hazards exist.


Guilin Refine Medical Instrument Co., Ltd.

No.8-3, Information Industrial Park, High-Tech Zone, Qixing District, Guilin, Guangxi, 541004, P.R. China

MedPath GmbH
Mies-van-der-Rohe-Strasse 8, 80807 Munich, Germany

Shelf life: 2 Years

Applied standards: GB 14866-2006 / EN166:2001 / ANSI/ISEA Z87.1-2015
AS/NZS 1337.1:2010 / CSA Z94.3-2015

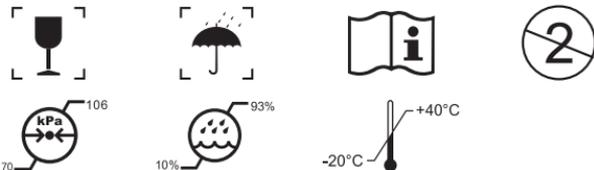

Symbol instruction

Symbol	Instruction	Symbol	Instruction
	Warning, Caution and Important ! Check the Instruction Manual		EU Representative
	Date of manufacture		Manufacturer
	Recovery		Handle with care
	Keep dry		Consult the accompanying documents
	Atmospheric pressure for storage		Single-Use
	Temperature limitation for storage		Lot number
	Humidity limitation for storage		MDEL certification
	According to ANSI/ISEA Z87.1-2015: D3-Special use, used to prevent Droplet / splash		CE certification
	EAC certification		
	According to EN 166:2001, REFINE - manufacturer 1 - Optical class (Class I) S - Symbol for increased robustness N - Symbol for Resistance to fogging of oculars EN 166 - Product Standards 3 - Field of use (Liquid Droplet/splash) S - Symbol for Increased robustness		

RF-MIB-B001
Edition: V1.4
Modify: 20200709



6 971560 45089 2