




The ABCs of Connex® ProBP™ 3400 Digital BP Device

<p>A</p> <p>Age Ranges</p>	<p>Not intended for use on younger than 3 years old. No mode changes required for use on Pediatric or Adult patients. Use appropriate cuff size for NIBP.</p> <p>Welch Allyn NIBP Cuff Range – Size 8 thru Size 13</p>
<p>B</p> <p>Battery Info</p>	<ul style="list-style-type: none"> Operates on one lithium-ion rechargeable battery Prior to initial use, charge for 6 hours. When fully charged, battery delivers approximately 100 readings. Takes approximately 6 hours to fully re-charge a dead battery.
<p>C</p> <p>Cleaning</p>	<p><u>Device</u> The following agents are compatible with the monitor:</p> <ul style="list-style-type: none"> Cavi-Wipes™ Sani-Cloth® Plus 70% isopropyl alcohol 10% chlorine bleach <p>Apply solution to cloth and wipe down monitor; never spray solution directly on monitor. Allow monitor to dry 10 minutes before use. Rinse off bleach with a cloth dampened with water. Note: Disinfect according to your facility's protocols and standards or local regulations.</p> <p><u>Accessories</u> NIBP hose/reusable cuffs: Wipe down with mild detergent solution. The same agents used to clean the device can be used on the stand and mounting accessories.</p>
<p>D</p> <p>Defaults</p>	<p>To change the default settings:</p> <ol style="list-style-type: none"> From the HOME screen, press the Right navigation button. Settings is highlighted. Press the Select  button. The Settings menu displays. Use Up or Down Navigation buttons to highlight the Unit of measure, Pressure preset, or Advanced menus. Press the Select  button to confirm. Press the Return button to exit screens.
<p>E</p> <p>End Use</p>	<p>To turn device off, press the Power  button. After 2 minutes of inactivity, the device goes into Standby mode. The screen will go blank; Press any button to bring the ProBP 3400 out of Standby mode. When running off battery power the ProBP 3400 automatically powers off when not used for 30 minutes . When the device is connected to external power, the device will automatically power off after three hours of non-use.</p>

Source: DFU