



HEAD OFFICE

Owen Mumford Ltd
Brook Hill, Woodstock
Oxfordshire OX20 1TU, United Kingdom
T : +44 (0)1993 812021
E : info@owenmumford.co.uk

UK / IRELAND SALES & MARKETING

Owen Mumford Ltd
Primsdown Industrial Estate
Worcester Road, Chipping Norton
Oxfordshire OX7 5XP, United Kingdom
T : +44 (0)1608 645555
E : info@owenmumford.co.uk

NORTH AMERICA

T : +1 770-977-2226
E : info@owenmumfordinc.com

FRANCE

T : +33 (0)2 32 51 88 70
E : fr.info@owenmumford.com

GERMANY

T : +49 (0)6026/97750
E : de.info@owenmumford.com

CHINA

T : +86 (0)21 6032 3528
E : cn.info@owenmumford.com

MALAYSIA

T : +60 (0)7 553 9617
E : my.info@owenmumford.com

MEXICO

T : +52 (55) 3733 8759
E : mx.info@owenmumford.com

owenmumford.com

OUR ACCREDITATIONS INCLUDE:

- ISO13485 Certification
- Medical Device Directive 93/42/EEC
- CMDCAS 13485 (Canadian Health Authority)
- Environmental Management Systems ISO14001



© Copyright Owen Mumford 2016
AHVATUTTHCPLT/MCC0116/02



Tiny feet deserve
a TinyTouch™



Unistik® TinyTouch™

Single use heel incision device



More reason to smile

owenmumford.com

Unistik® TinyTouch™

Tiny feet deserve a TinyTouch™

Neonatal capillary blood testing presents many challenges for healthcare professionals – producing an adequate sample volume in a single incision, yet minimising any discomfort to the baby and distress to the family.

This heel incision device features an alignment guide designed to accurately place the device on the infant's heel.

With penetration depths of 0.85mm for preterm neonates and 1.00mm for full term neonates our **Unistik® TinyTouch™** has been designed to reach superficial blood vessels without penetrating deep enough to reach deep dermal pain fibers. This type of device results in less pain for your tiny patients.¹

Activation button.

After use, the activation button remains depressed and the **blade permanently retracts**, helping to prevent re-use or injury.

Sterility cap.


The easy-grip, quick-release pull tab reduces the stages required for use, while keeping the blade sterile.

Alignment guide.

Clearly visible from underneath and either side of the device, the alignment guides **enable precise placement**.



How to use Unistik® TinyTouch™

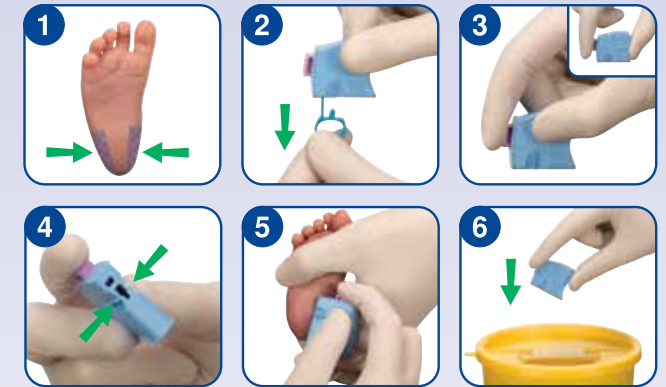

Instructions for Use

In accordance to your institution's guidelines, gently warm the infant's heel and cleanse the outer side of the heel where you will perform the incision (1).

Remove the sterility tab by pulling straight off (2).

There are two ways to grip the device. The preferred position is with thumb and finger gripping on the wider side. Alternatively, you may grip with your thumb against the narrow side (3).

Using the alignment guide (4), align the blade slot with the incision site (5), then activate the button. Blade will permanently retract after use. Dispose of in a suitable sharps container (6).

To express blood, gently apply pressure at the top and bottom of the incision site.

Product range



Premie

Depth: 0.85mm
Length: 1.75mm
Box of 50: AT2004



Full Term

Depth: 1.00mm
Length: 2.50mm
Box of 50: AT2005

¹. Shah, V. et al. (2003) Evaluation of a New Lancet Device (BD Quikheel) on Pain Response and Success of Procedure in Term Neonates. *Archives of Pediatrics and Adolescent Medicine*, 157: 1075-1078.