


Owen Mumford Proudly Introduces

Unistik[®] TinyTouch[™]



Heel Incision Device

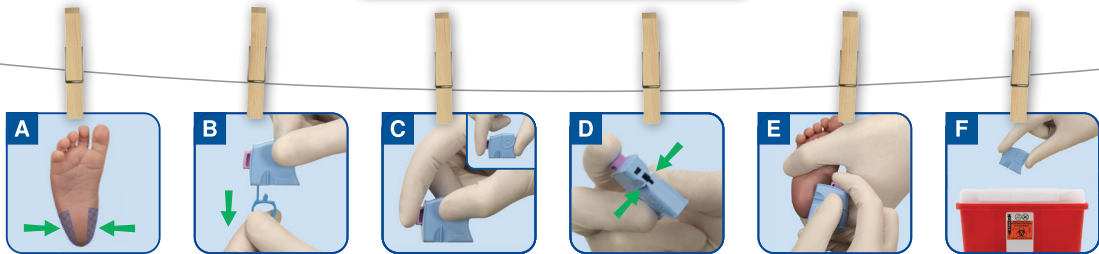

Instructions For Use

In accordance with your institution's guidelines, gently warm the infant's heel and cleanse the outer side of the heel where you will perform the incision (A). Remove the sterility tab by pulling straight off (B). There are two ways to grip the device. The preferred position is with thumb and finger gripping on the wider side. Alternatively, you may grip with your thumb against the narrow side (C). Using the alignment guide (D), align the blade slot with the incision site (E), then activate the button. Blade will permanently retract after use. Dispose of safely in a suitable sharps container (F). To express blood, gently apply pressure at the top and bottom of the incision site.



Owen Mumford Proudly Introduces

Unistik™ TinyTouch™



Heel Incision Device



Because Tiny Feet Deserve a TinyTouch

Tiny feet can offer big challenges when it comes to neonatal capillary testing. Unistik Tiny Touch Heel Incision Device is designed to deliver quality blood sampling with reduced pain and discomfort, for happier babies and healthcare workers.



Preemie

Box of 50
Color: White
Incision Depth & Length
0.85mm x 1.75mm
Order Number: AT1200



Full-Term

Box of 50
Color: Blue
Incision Depth & Length
1.0mm x 2.5mm
Order Number: AT1250