



Important Instructions

Read carefully before use




The HCG-801 records your heart signal and evaluates on several heart conditions


- Please note that the HCG-801 device **cannot replace the interpretation** of the heart signal by a **medical expert**.
- **Only use** the HCG-801 **on recommendation** of or after consulting your **doctor**.
- The HCG-801 **cannot detect all possible heart conditions**.
- When you **experience symptoms** related to heart disease **always consult your doctor** even if “stable waveform” is displayed.
- The HCG-801 is **not designed or intended** to be used in **acute or critical situations**.
- **Self-measurement is not the same as medical diagnosis. Never attempt to interpret** the results and **start or modify your treatment or medication** without consulting your doctor.
- The HCG-801 takes a **snapshot of about 30 seconds** of your heart signal. Your heart condition can change before and after you take a reading. If you notice any change in your condition always consult your doctor for medical advice.

Common Causes of Inaccurate Measurement Results

The following situations can result in inaccurate measurements or errors:

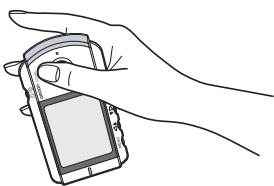


The index finger is not placed closely on the electrodes.

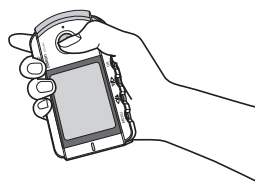

The unit is held incorrectly.



The unit is placed over clothing.

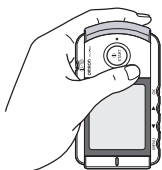

The unit is held in the left hand. Always hold the unit in your right hand during the measurement.



The rear side of the unit is placed against the chest.



The chest electrode is against the finger and the finger electrodes against the chest.