



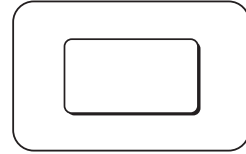

Mölnlycke®

Mepore® Ultra

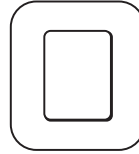

STERILE EO

UK
CA 0086

CE 2797

Revised 2025-04

327403-00

PD-779882 rev. 1

Master PD-549460 rev. 02



Mölnlycke Health Care AB
Entreprenörstråket 21
SE-431 53 Mölndal, Sweden

www.molnlycke.com



Single use



See instructions for use




Do not use if package is damaged



Medical Device



Single sterile barrier system




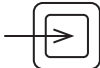


Wound pad size



Showerproof

www.molnlycke.com/symbols

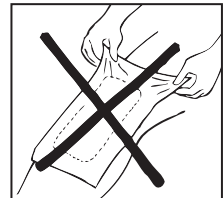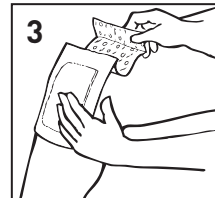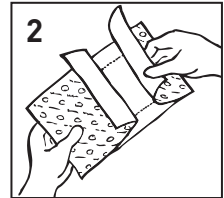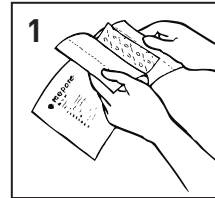

Mepore® Ultra

Absorbent showerproof adhesive dressing

Product description

Mepore Ultra is a sterile, self-adhesive and absorbent dressing.

Mepore Ultra consists of:

- an absorbent wound pad with an integrated non-adherent wound contact layer
- an adhesive border with water based acrylic adhesive.
The adhesive is skin friendly and free from organic solvents.
The adhesive area is covered with two overlapping release liners enabling aseptic application.
- an outer film which is breathable but waterproof, providing a barrier to external contaminants

Dressing material content:

Acrylic adhesive, viscose nonwoven, polyester nonwoven, polyurethane film

Indications for use

Mepore Ultra is designed for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

Precautions

- Do not use Mepore Ultra on patient with known hypersensitivity to acrylic adhesives or other ingoing materials/components of the product.
- Do not use on fragile skin. If used on fragile skin it may cause skin irritation.
- Do not stretch the dressing when applying.
- If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if inner package (sterile barrier) is damaged or opened prior to use. Do not re-sterilise.

Instructions for use

Mepore Ultra can be used by lay persons under supervision of health care professionals.

1. Cleanse the wound in accordance to clinical practice. Dry the surrounding skin thoroughly. Usage of moisturizers, gels or ointments on the skin surrounding the wound may affect the adhesion and should be avoided.
2. Remove the release liners and apply the sticky side to the skin without stretching the dressing. Ensure that the absorbent pad covers the wound.
3. Gently press the border to ensure proper adhesion to the skin.
4. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

Care should be taken to ensure that Mepore Ultra is not applied under tension, to prevent shear forces causing damage to the skin. When applying over joints be careful to apply in such a way that it does not impair flexibility of the joint. After some surgical interventions excessive swelling may occur. In such case it could be better to use a more flexible dressing in order to minimise skin tension.

Mepore Ultra should be change before the dressing is fully saturated, at signs of leakage or as indicated by clinical practice. Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Mepore Ultra it should be reported to Mölnlycke Health Care and to your local competent authority.

Mepore® is registered trademark of Mölnlycke Health Care AB.