




asma-1 Respiratory Monitor

Guide for Healthcare Professionals




Testing Subject

1 Insert Mouthpiece and 1 Switch On

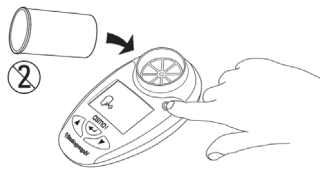

2 Blow 3 times

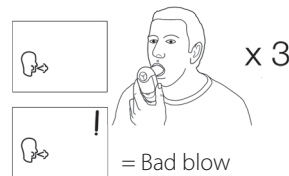

3 Best blow



Set PEF Personal Best

1 1 Switch On



2 ▼ + ▲ hold for 3 seconds



3 ▲ Increase PEF



4 ← Set PEF best



Press ← ← to exit or

Set FEV₁ Best

5 ▲ Increase FEV₁



6 ← Set FEV₁ best



Note: Zones set at 80% and 50%

EN ISO 23747:2007

Specifies accuracy: ± 10% PEF
Resistance to flow: < 0.35 kPa/L/s

Vitalograph asma-1

Accuracy: ± 5% PEF
± 3% FEV₁
Resistance to flow: < 0.15kPa/L/s

Review History

1 1 Switch On



2 ← Hold for 3 seconds



3 ▲ or ▼ Review History



4 ← Return




Delete History

1 Switch on



2 + hold for 10 seconds

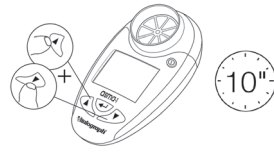

Cleaning & disinfecting:

The outer surfaces should be cleaned every week, more often if necessary. The use of a disposable disinfecting wipe is recommended, with special attention to the mouthpiece area.

Modify Green Zone

1 Switch On



2 + hold for 3 seconds



3 or Upper Boundary



4 Set Upper Boundary



Modify Other Zones

5 or Lower Boundary



6 Set Lower Boundary



7 or Mid Boundary



8 Set Mid Boundary




Note: 3-Zones now set.
Press to exit or for 4-Zones...

Default Zone Settings


1 Switch On



2 + + hold for 10 seconds

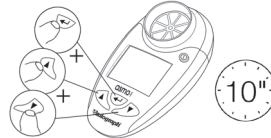

3 Switch Off

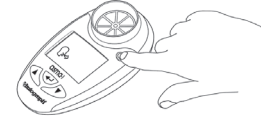

? www.vitalograph.co.uk

Vitalograph[®]

Your respiratory partner

UK Sales
Vitalograph Ltd.
Maids Moreton Buckingham
MK18 1SW England
Tel: 01280 827110
Fax: 01280 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.co.uk

Export Sales
Vitalograph Ltd.
Maids Moreton Buckingham
MK18 1SW England
Tel: +44 1280 827120
Fax: +44 1280 823302
e-mail: sales@vitalograph.co.uk
www.vitalograph.eu

Vitalograph GmbH
Jacobsenweg 12
22525 Hamburg
Germany
Tel: (040) 547391-0
Fax: (040) 547391-40
e-mail: info@vitalograph.de
www.vitalograph.de

Vitalograph Inc.
13310 West 99th Street
Lenexa
Kansas 66215 USA
Tel: (913) 888 4221
Fax: (913) 888 4259
e-mail: vitcs@vitalograph.com
www.vitalograph.com

Vitalograph (Ireland) Ltd.
Gort Road Business Park
Ennis
Co Clare Ireland
Tel: (065) 6864100
Fax: (065) 6829289
e-mail: sales@vitalograph.ie
www.vitalograph.ie