

New
and
Improved

3M™ Cavilon™ Durable Barrier Cream

The Gold Standard

for protection of vulnerable skin

Cavilon durable barrier cream now provides an improved moisturising barrier for skin at risk from breakdown.

Maintaining patients' skin integrity is our number one priority, so we have listened to the experts in this field in order to improve Cavilon durable barrier cream and set the gold standard for skin protection across the UK.

3M has upgraded Cavilon durable barrier cream to provide:

- Improved moisturisation and barrier protection¹
- Perfume and parabens free
- Safe to use on injured skin²



Code changes

	Previous code	New code
Durable Barrier Cream 28g tube	3391E	3391G
Durable Barrier Cream 92g tube	3392E	3392G
Durable Barrier Cream 2g sachet	3392S	3392GS

If you would like more information about Cavilon skin care products, to receive product samples or to arrange a 3M Representative visit, call **0800 616066 (answer phone)** or visit **www.cavilon.co.uk**.

References:

1. 3M Data on File 2014, Validation of Cavilon Next Generation Durable Barrier Cream; Relating to improved barrier function, Lab. report no. PD-PRO-RPT-07-235012
2. 3M Data on File 2014, HClPT Irritation Study, Lab. report no. CLIN-RPT-FINAL-INV-US-05-238099

PREVENTION

3M™ Cavilon™ Wipes

Available from NHS Supply Chain

3M Code	Description	Size	Wipes/pack	Packs/Case	NHS Code
9274	Continence Care Wipes	20cm x 30cm	8	12	ELY566
9272	Bathing & Cleansing Wipes	20cm x 30cm	8	12	ELY567



Green to Clean

3M™ Cavilon™ Durable Barrier Cream

- As a barrier against irritation from body fluids
- Protection from damage associated with incontinence
- As a moisturiser for severely dry skin
- Purple packaging - think Purple for Prevention

Application Techniques - Following Patient Assessment

1. Skin should be clean and dry prior to application of Cavilon durable barrier cream.
2. Apply Cavilon durable barrier cream sparingly (in pea-size amounts) to cover entire affected area. Repeat as necessary.
3. If the skin feels oily, too much has been applied. Re-application is recommended at every third episode of incontinence.
4. When used to moisturise severely dry skin apply daily or as needed.

3M™ Cavilon™ Durable Barrier Cream available from NHS Supply Chain

3M Code	Description	Size	Presentation	NHS Code
3391G	Durable Barrier Cream	28g per tube	12 tubes/case	ELY571
3392G	Durable Barrier Cream	92g per tube	12 tubes/case	ELY568
3392GS	Durable Barrier Cream	2g sachet	20 sachets/box	ELY569



Purple for Protection

Available on Drug Tariff Part IXC, Skin Fillers and Protectives from January 2015

3M Code	Description	Size	Presentation	Link Code	PIP Code
3391G	Durable Barrier Cream	28g per tube	1 tube/box	CAV22R	301-7480
3392G	Durable Barrier Cream	92g per tube	1 tube/box	CAV20C	277-1079
3392GS	Durable Barrier Cream	2g sachet 20	sachets/box	CAV21V	288-2272

TREATMENT

3M™ Cavilon™ No Sting Barrier Film

- As a primary barrier against irritation from body fluids
- A protective barrier against adhesive from wound dressings and surgical tapes
- Peri-wound protection from exudate damage
- Protection from damage caused by friction
- Blue packaging - think Blue for use on Broken skin

Application Techniques - Following Patient Assessment

1. Skin should be clean and dry prior to application of Cavilon no sting barrier film.
2. Apply a uniform coating of film over the entire treatment area; allow to dry (approximately 30 seconds). When spray is appropriate, hold the spray nozzle 10 to 15cm from the skin and apply in a smooth even coating over the entire treatment area, whilst moving the spray in a sweeping motion. The spray is particularly beneficial when the treatment area is painful.
3. If an area is missed, reapply to that area only after the first application has dried (approximately 30 seconds).
4. If Cavilon no sting barrier film is applied to an area with skin folds or other skin-to-skin contact, make sure that the skin contact areas are separated and allow the coating to dry before returning to normal position.
5. Re-application is recommended at least every 48-72 hours (under normal use) or more frequently if needed, in severe incontinence re-apply every 24 hours.

3M™ Cavilon™ No Sting Barrier Film available from NHS Supply Chain

3M Code	Description	Size	Presentation	NHS Code
3343E	Foam applicator (sterile)	1 ml	25 applicators/box	ELY038
3345E	Foam applicator (sterile)	3ml	25 applicators/box	ELY039
3346E	Spray Bottle (pump action)	28ml	12 bottles/box	ELY040

Available on Drug Tariff Part IXC, Skin Fillers and Protectives

3M Code	Description	Size	Presentation	Link Code	PIP Code
3343P	Foam applicator (sterile)	1 ml	5 applicators/box	CAV16U	252-8941
3345P	Foam applicator (sterile)	3ml	5 applicators/box	CAV17Q	252-8958
3346P	Spray Bottle (pump action)	28ml	1 bottle/box	CAV14F	252-8966



Blue for Broken



3M Health Care Limited
3M House
Morley Street
Loughborough
Leicestershire LE11 1EP
Tel: (01509) 611611
Fax: (01509) 237288
www.cavilon.co.uk

3M Ireland
The Iveagh Building
The Park
Carrickmines
Dublin 18, Ireland
Tel: 00 353 (01) 2803555
Fax: 00 353 (01) 2803509

www.cavilon.co.uk

3M and Cavilon are trademarks of the 3M Company.
© 3M Health Care Limited 2014.