## COOL'S EB04.003

SUGGESTED USE | SUGERENCIAS DE USO

### FRONTAL AND BACK PANEL | PANEL FRONTAL Y TRASERO

- 1 Washable waterproof material.
- 2 Multipurpose pocket.
- 3 Transparent upper pocket for documents.
- 4 Frontal transparent pocket for personal belongings.
- 5 Thermometer.
- 6 Horizontal compartment to organize cans urine samples.

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### MAIN COMPARTMENT | COMPARTIMENTO PRINCIPAL

- 1 Three detachable vertical compartments with handle for easy removal and portability.
- 2 Front flat transparent pocket for identify cards.
- 3 The vertical compartments are compatible with PVC, metal and polystyrene racks, and include two internal pockets for cold gel.
- 4 Light and compact rack, made of sponge, with capacity of up to 44 sample tubes of differents sizes (6 units).

In compliance with UN3373 regulation for the transport of biological samples, each rack includes a watertight bag with double zip lock to avoid contamination and a sponge for absorbing leaks.

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#### \* TROLLEY BAND:

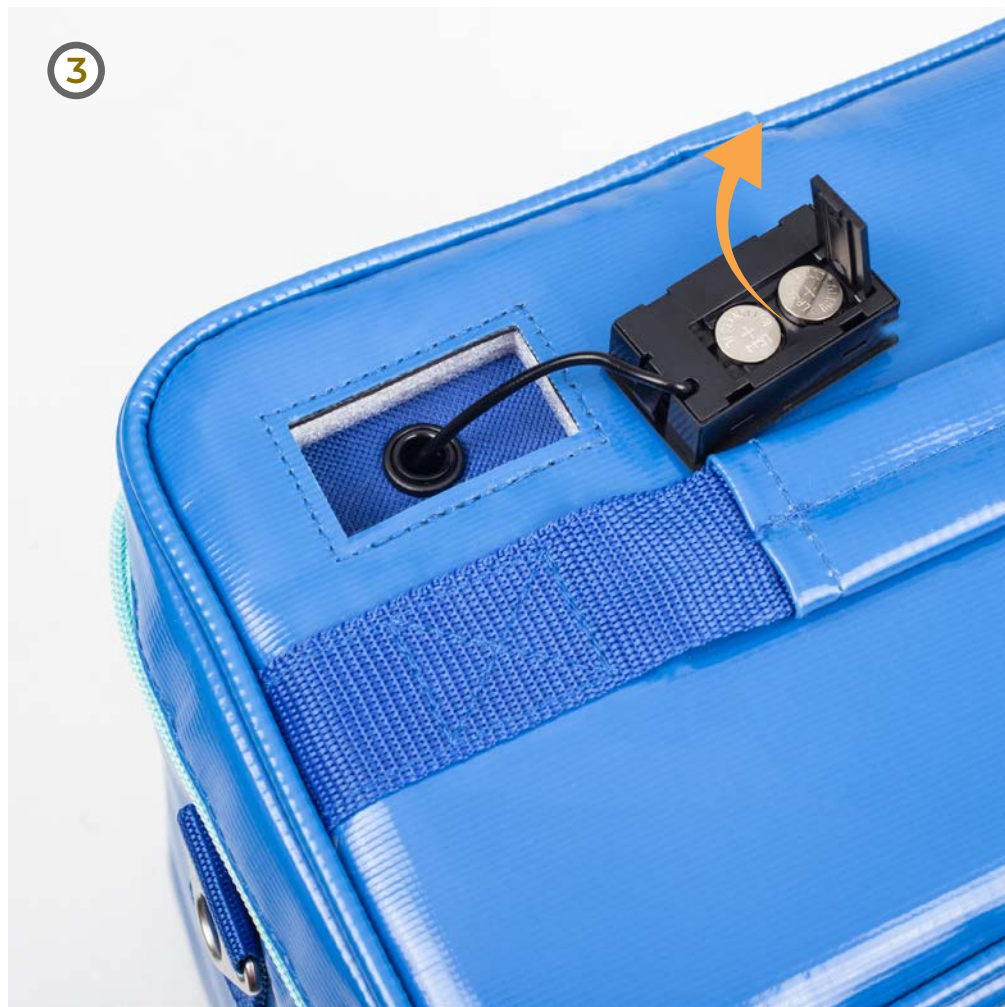
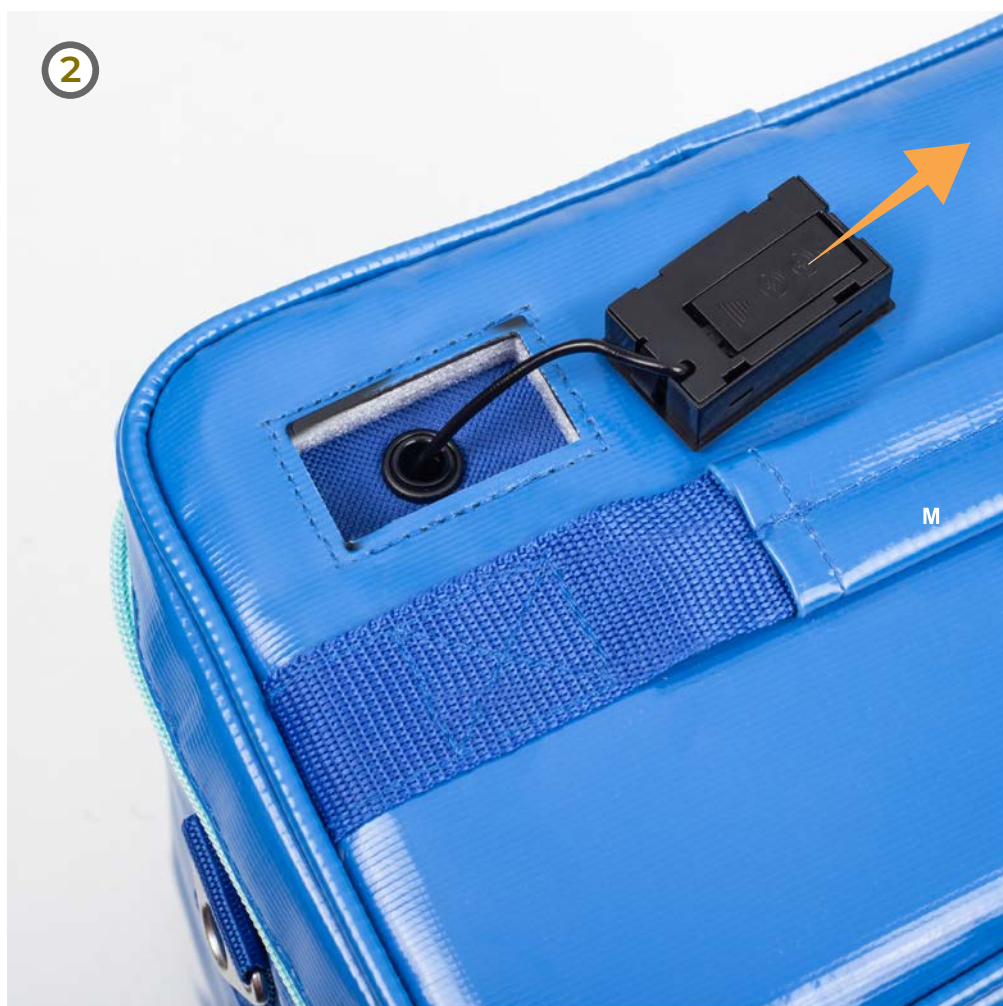
Possibility to incorporate an external trolley structure, making the transport easier. Our reference Carry's EB09.023 is robust, light and easy to fold, and thus suitable for this purpose.

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### BATTERY REPLACEMENT | REEMPLAZO DE PILAS

- 1 Remove the LCD module.
- 2 Slide the battery door in the direction indicated by the arrow.
- 3 Open the compartment and replace the exhausted batteries.
- 4 Close the hatch again and insert the module again until it clicks into place.

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