



Training – 2020

# HEINE DERM APP

# HEINE DERM APP



Strong Digital Added Value to our premium Dermatoscopes!

- Training Meeting Rules
- What value do we want to communicate regarding the App? Benefits and Features!
- Questions and Feedback
- How to operate the App
  - Live – hands on guiding through the App
- Questions and Feedback
- Competitive Options
- Next steps and timing?



HEINE Derm app

# EXCLUSIVE HEINE DERM APP (APPLE)



Selling Line, Features & Benefits, top 5 arguments

**Easy management of digital photos, comments and data.**

**Storage and management** of dermatoscopic digital images with an iPhone\*.

**Bodymap:** easy allocation of photos.

**Addition of comments,** risk levels, diagnoses and therapies.

**Intuitive workflow** through a simple user interface.

**Easy comparison** of follow-up images.



HEINE Derm app

# EXCLUSIVE HEINE DERM APP



## Features & Benefits

**Extremely sharp detail and realistic colour rendering:** thanks to the App optimized adjustment of the camera system in combination with the LED<sup>HQ</sup> of HEINE dermatoscopes.

**Zoom function:** up to 40x digital magnification.

**High-resolution image quality storage.**

**Automatic date capture.**

**Quick access** to patient records, images and information.

**Sharing function\*\*:** The images can easily be forwarded via e-mail or messenger services (WhatsApp, Telegram etc). For example for a second opinion, a teleconference or for referral to a specialist, but also to patient management systems (PMS).



HEINE Derm app

# EXCLUSIVE HEINE DERM APP



## Features & Benefits

**Data control** through local storage\*\*\* on the iPhone.

**Increase patient loyalty:** At the end of the examination, images and further treatments can be discussed immediately and vividly.

**Klick & Work:** easy connection of mobile phones to HEINE dermatoscopes with matching HEINE accessories.

**One software** for the HEINE Dermatoscopes DELTA 30, DELTAone, iC1 and NC2.

\* The Apple iPhone is not included. Apple, the Apple logo, iPhone and iPad are trademarks of Apple Inc. registered in the U.S. and other countries. App Store is a service mark of Apple Inc. HEINE recommends the use of an iPhone from version 6.

\*\* All according to local legislation and/or the patient's consent.

\*\*\* Backups should be made according to country-specific data protection regulations and individual security needs.



## HEINE Derm app

# EXCLUSIVE HEINE DERM APP




## How to operate the App

You can start the process in two ways.

- Start with setting up a patient (Patients Tab)
- Start with taking a picture (Examinations Tab)

### The patients tab

- Gives you the overview on the patient list
- Once you tab on the patient the body map appears.
- Above the body map you will find 3 Tabs
  - Body map
  - List
  - Data
- You can take new pictures of a new mole directly from the body map tap or the List Tab.
- The same applies for a re-take of an existing mole
- The data tab enables you to make changes to the data of the patient.

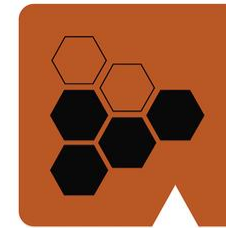

HEINE Derm app



# EXCLUSIVE HEINE DERM APP



## How to operate the App

### The examinations Tab:

Gives you an overview of pictures taken:


- Today – listed chronologically by patient name
- Past – Also listed chronologically by patient name
- Incomplete – you can find all pictures that have no patient related to it

### How to export pictures?

Only single pictures to be exported.

Go to patient or Examination – pick the picture – click on the export icon and select the media you want to send it with:

- ▶ Email
- ▶ Any installed messangers that allow the sending of pictures

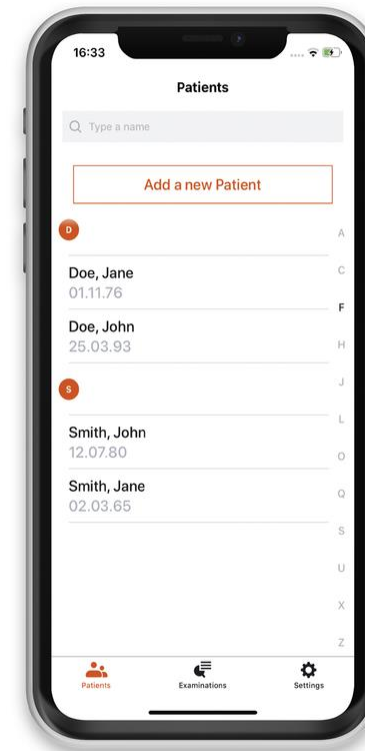

# EXCLUSIVE HEINE DERM APP




## How to operate the App

### The Settings tab:

Basically only interesting for the setting of patient detail starting page.

- ▶ You can choose between:
  - Body map view
  - List view
  - Info View

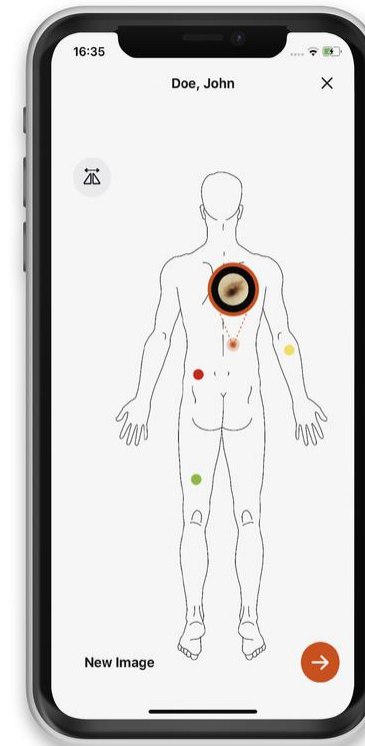





THANK YOU!

**HEINE** QUALITY  
MADE IN GERMANY

HEINE Optotechnik GmbH & Co. KG  
Dornierstr. 6 | 82205 Gilching | Germany  
Tel. +49 (0)8152 38-0 | [info@heine.com](mailto:info@heine.com) | [heine.com](http://heine.com)