



HOW TO CONTROL LIFE-THREATENING BLEEDING WITH MAG-TQ


ONE OR TWO HANDED APPLICATION

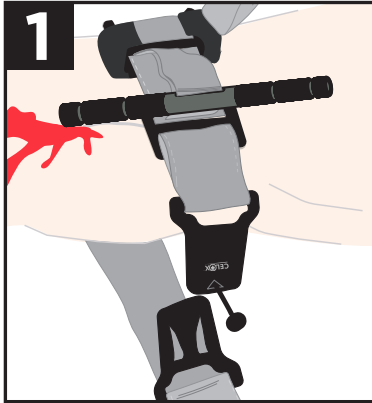


1
Apply 2-3in (5-7.5cm) above bleeding site.

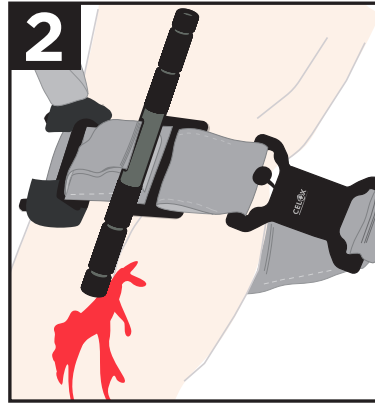


2
Connect magnetic buckle.

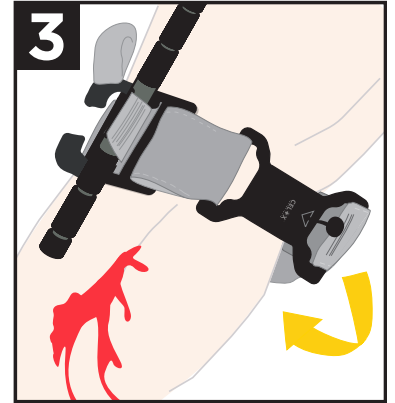


3
Pull tight.

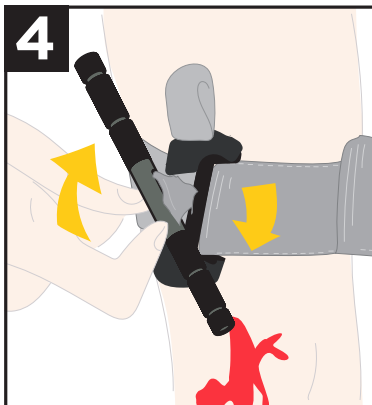

4
Twist windlass until bleeding stops*.

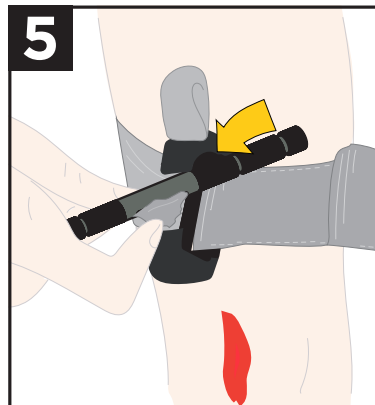


5
Secure windlass in gate.

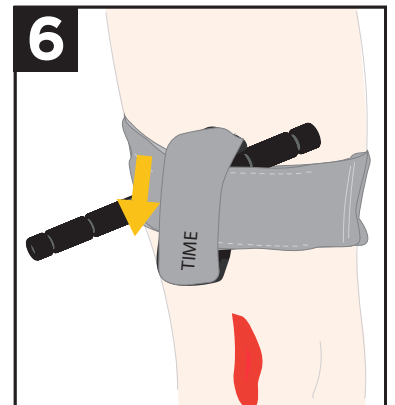


6
Route remaining strapping, close with time tab and record time applied.

REMOVAL INSTRUCTIONS

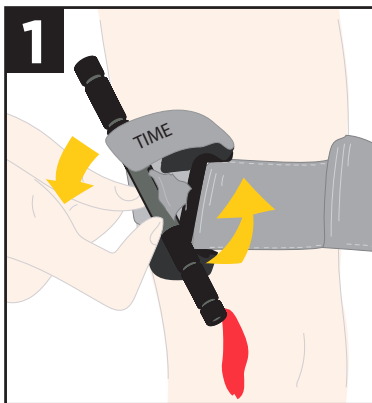

1
Release time tab and untwist windlass rod.

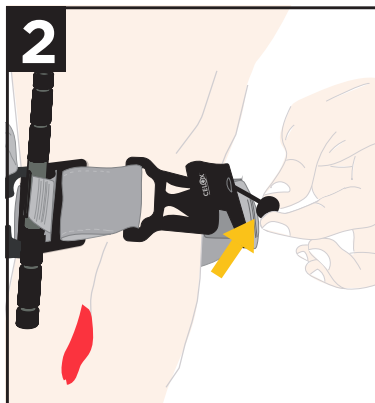

2
Pull the tab upwards until the magnetic buckle disconnects. Dispose of as clinical waste after use.



*if bleeding is not controlled, tighten or reposition or apply 2nd MAG-TQ above the first.

For full details, warnings and removal instructions refer to manufacturer's instructions for use.

**FOR MORE INFORMATION VISIT
CELOXMEDICAL.COM**