

CONNEX VITAL SIGNS MONITOR 6000 SERIES WITH CONNEX®VM QUICK REFERENCE CARD - PUSH DATA FROM DEVICE WORKFLOW



Partners in Care Services
SUPPORT | EDUCATION | PROFESSIONAL

WelchAllyn®

To Capture Vitals:


- Turn on the CVSM 6000 Series device.
- Touch the clinician area. Scan the **Clinician's ID**.
Touch **Select**.
- From **Home** screen, scan the patient's barcode, OR touch **Patients, List** and then the patient. Touch **Select**.
- Capture the vital signs.

NOTE: if available, SpHb numeric value will automatically transfer in.

NOTE: For **Spot Check profile:** Remove the SpO2 sensor to save the reading. For **Monitoring profile:** Keep the SpO2 sensor on to save the reading.

- Touch HEIGHT, WEIGHT, PAIN or RR (respiratory rate) to enter values. Touch **OK** when done.
- To enter BP site, cuff size, patient position, SpO2 method, concentration or flow rate, touch **Patients** tab and then **Modifiers**.
- When readings are complete, touch **Save**.

Clinician area

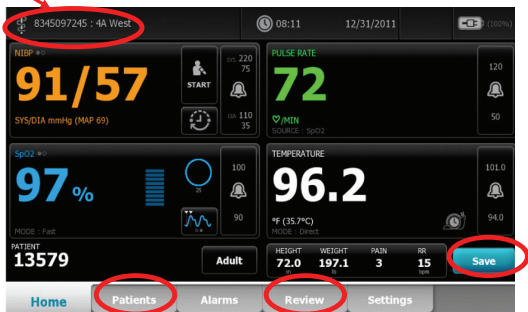

To Push Data from Device:

- Attach the device to a PC with a USB cable.
- Touch the **REVIEW** tab.
- Touch readings you want to send (a ✓ will appear next to selected items). Touch **Send**. Touch **OK** to confirm.

NOTE: Depending upon configuration, once you touch **Send**, data is sent to Connex®VM and the memory on the device is **Cleared**.