

CONNEX VITAL SIGNS MONITOR 6000 SERIES WITH CONNEX®VM QUICK REFERENCE CARD – WIRELESS WORKFLOW



Partners in Care Services
SUPPORT | EDUCATION | PROFESSIONAL

WelchAllyn®

To Capture Vitals:

- Turn on the CVSM 6500 device.
- Touch the clinician area. Scan the **Clinician's ID**. Touch **Select**.
- From **Home** screen, scan the patient's barcode, OR touch **Patients, List** and then the patient. Touch **Select**.
- Capture the vital signs.


NOTE: if available, SpHb numeric value will automatically transfer in.

NOTE: When using the **Monitoring profile**, in order to capture the pulse oximetry value you must leave the SpO2 sensor on when you press **Save**.

- Touch HEIGHT, WEIGHT, PAIN, RR to enter values. Touch **OK** when done.
- To enter patient position, BP site, cuff size, SpO2 method, concentration, or flow rate, touch **Patients** tab and then **Modifiers**.
- When readings are complete, touch **Save**.

NOTE: This will send your reading when **autosend** is **enabled**.

Clinician area

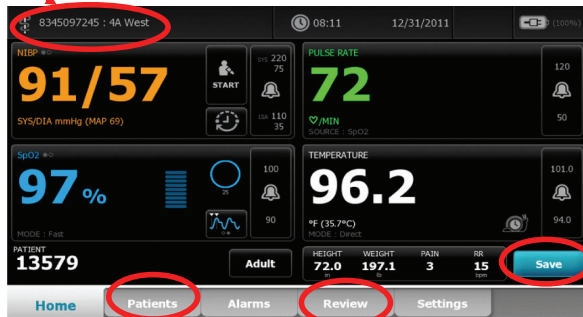

To send readings taken during Intervals:

- Touch the **REVIEW** tab.
- Touch the readings you want to send, then touch **Send**. Touch **OK** to confirm send.

NOTE: Depending upon configuration, once you touch Send, data is sent to Connex®VM and the memory on the device is **Cleared**.