



TVAC

Manual Suction Device

REF TM-909



1-855-TyTekMD (898-3563) TyTek Medical Inc. 8904 Beckett Rd., West Chester, OH 45069, US
www.tytekmedical.com

Description

TVAC is a compact, single-use, manual suction device intended for clearing mucus, blood, vomitus and other foreign substances from the upper airway. As TVAC is manually operated it is not dependent on batteries or any other external power source. The TVAC is made of a rigid plastic suction tube, a suction bulb and a collection bag for aspirated liquids. Backflow of aspirated contents is prevented by one-way valves. Its rugged construction allows TVAC to be stored for extended periods of time and instantly made ready for use. Product does NOT contain Natural Rubber Latex.

Caution

TVAC should only be used by personnel who have received proper training on procedures for suctioning an oral airway. Improper use of TVAC could cause injury. Use TVAC as directed by your EMS authority or by a physician. TVAC is NOT designed to be used on children under age eight.

Instructions For Trained Medic

- Remove TVAC from its package. Keep the package nearby and available for use as the disposal package.
- Grasp the bulb with one hand and the suction tube with the other. IMPORTANT: Pull out the suction tube until it is fully extended.
- Use finger sweeps to remove foreign matter from the patient's airway.
- Grasp the suction bulb and squeeze it firmly.
- Keep the bulb in the fully squeezed position as you insert the suction tube into the back of the patient's throat, being careful not to insert the tube farther than you can see.
- Release the already-squeezed bulb, enabling it to remove liquid in the patient's airway.
- Continue to squeeze and release the bulb to remove all liquids into the collection bag. A one-way valve will prevent backflow of collection fluids.
- Once the patient's airway is cleared with TVAC, take any other necessary steps to maintain the cleared airway.
- See disposal section to properly dispose of device.
- Note: Any serious incident that occurs in relation to the device shall be reported to the manufacturer and the competent authority in which the user/patient is established.

Body Substance Isolation Alert

Use of this product may result in your contact with contaminated body fluids. Use appropriate body substance isolation measures along with personal protection measures such as gloves and eye protection in order to prevent contact with contaminated fluids.

Disposal

TVAC is for single-use only. It is not re-usable and is designed for disposal after use. Do not try to clean or reuse this device. It may be used once, and then must be properly disposed. Dispose of TVAC in a way that assures body substance isolation of potentially infectious substances. To dispose of TVAC, push the tube back into the bulb far enough so that the unit and contained fluids will fit into the original package. Seal the original package with its easy-seal feature. Remove the peel-away label on the package to expose the BIOHAZARD label underneath. Dispose of per your local biohazard waste regulations.

Storage

This device should be kept dry and stored in temperatures between 10° F and 140° F.

Limited Warranty

The products and components contained in this package are medical devices intended solely for use by appropriately trained healthcare professionals. TyTek Medical Inc. warrants that the products and components are free from defects in material and workmanship and fit solely for use as medical devices for the specific purpose described in the manual by trained healthcare professionals for a period of 12 months from the date of shipment by TyTek Medical Inc. Any use of the products or components beyond the use the products or components voids this limited warranty. EXCEPT AS SET FORTH HEREIN, TYTEK MEDICAL INC. MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, REGARDING THE PRODUCTS OR COMPONENTS AND SPECIFICALLY DISCLAIMS THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

This item sold and to be used strictly per terms and conditions at www.tytekmedical.com.