



virustatic[®]
SHIELD

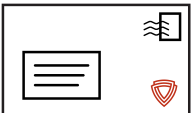
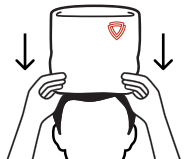
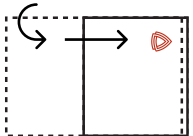
USER INFORMATION

For more information contact us on customer.service@virustaticshield.com
Visit our website at www.virustaticshield.com

Application instructions

Follow the simple instructions below for the correct way to fit the Virustatic Shield.

For more information on how to care for your Virustatic Shield including important information regarding do's and don'ts - please refer to the important information guide.

1 Your Virustatic Shield will arrive via the post 	2 Wash your hands before opening 	3 Remove Virustatic Shield from the envelope 	4 Rub the colour change icon on the security label 
5 Scan the QR code using any barcode app 	6 Check your purchase is a genuine product 	7 Carefully remove the shield from the wrapper Do not cut the fabric 	8 Discard packaging Recycle where possible 
9 Wipe your hands using the fabric of the shield Make sure hands are dry 	10 Pull the shield down over your head 	11 Use as a single layer for up to 98% virus protection 	12 Or double layer for a tighter, more effective fit for the smaller head 
13 Pull shield over your head with the fold at the bottom (when using folded) 	14 Cover mouth and nose, making sure of a tight fit to ensure maximum protection 	15 Best Practice Always wash hands before wearing and removing Shield Keep cool and dry when not in use 	16 Important information Do not iron Do not tumble dry Wash with water only Leave to air dry naturally 

Important information

Read the instructions below for the correct way to care for your Virustatic Shield.

The information below must be followed to ensure the protective coating is not damaged.
Failure to follow the instructions below will reduce the protective qualities of the fabric.

Do not machine wash
Hand wash in water only



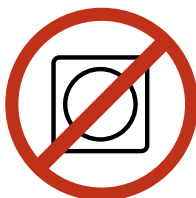
**Hand wash shield sparingly
& only when necessary**
Washable up to 3 times



Do not use detergents
Hand wash in water only



Do not tumble dry
Leave to air dry naturally



Do not iron
Heat damages the coating



**Do not cut or pierce
the fabric**



**Do not spray fresheners
or anti-bacterial
products on the fabric**



**Do not spray perfume
on the fabric**



Leave to dry naturally
Do not use a heat source

