

SHEFFSHIELD

PRODUCT INFORMATION & INSTRUCTIONS



Adjustable PPE Face Shield

SheffShield is a PPE device designed to protect against airborne liquid droplets of substances and mixtures of harmful biological agents which are hazardous to health and cause illness. This device provides full face and lateral protection, featuring a liquid-proof, transparent visor to provide protection to the user.

Note:

- This protective face shield is manufactured for COVID-protection only—it is not a PPE device for general use and shall not be used for purposes other than protection against COVID-19.
- The face shield may be used in conjunction with a respirator mask and/or additional eye protection such as glasses, and a gown.
- It is the responsibility of individual users to determine allergies to any materials used to construct this face shield; they should not use the product if allergic to the following:

 Headband polypropylene; Visor PET; Strap silicone rubber.
- This product is BSI Certified to PPE Regulation for Healthcare Professionals 2020/403.

Product Name: Full Face Visor Product Model: 3366-010

To assemble:

- Push two front headband mounting studs through middle two holes of visor;
- Stretch the headband arms outwards slightly and push side headband mounting studs through outer two visor holes;

ASSEMBLY

3. **Attach** elastic strap to the rear notch on each arm of the headband.






To fit:

- 1. Hold headband arm ends;
- 2. Place elastic at the back of the head;
- 3. **Pull** the headband forward into position;

FITTING

Ensure the elastic strap is at a comfortable position. If not, reposition elastic strap to a different notch.


REMOVAL

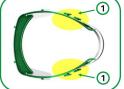




To remove:

- 1. Hold headband arm ends:
- 2. **Pull** the headband forward then up from the front of the face, releasing the elastic.

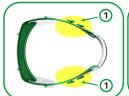

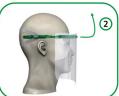

CLEANING & DISINFECTION

The face shield can be cleaned and disinfected using sodium hypochlorite, ethyl alcohol or isopropyl alcohol following suitable disinfection directions.

MAINTENANCE

Regular inspection of the face shield should be completed and if any defects found, the item **discarded**:

- Ensure the visor surface is free from scratches which could dangerously impair vision.
- 2. **Ensure** the headband and retaining strap are damage and wear free.

The face shield has a shelf-life of 3 years after manufacture and after this date should be **retired and destroyed**.

STORAGE & TRANSPORT

The face shield **should be stored** in a dry location away from sources of heat and strong sunlight.

Do not store with any objects on top of the face shield, or by stacking the shield such that the headband may become permanently deformed.

Certification: BSI's PPE for Healthcare Professionals 2020/403—Eye protection Technical Specification

Notified Body: BSI Assurance UK Ltd, Kitemark Court, Davy Avenue, Knowlhill, Milton Keynes, MK5 8PP, UK

Notified Body No.: 0086 BSI Certificate No.: 729058

9

Find us online



f facebook.com/SheffShield

For more information, please get in touch: Email: sheffshield@eadonconsulting.co.uk